




EUROLineScan

Automated evaluation of membrane-based test systems

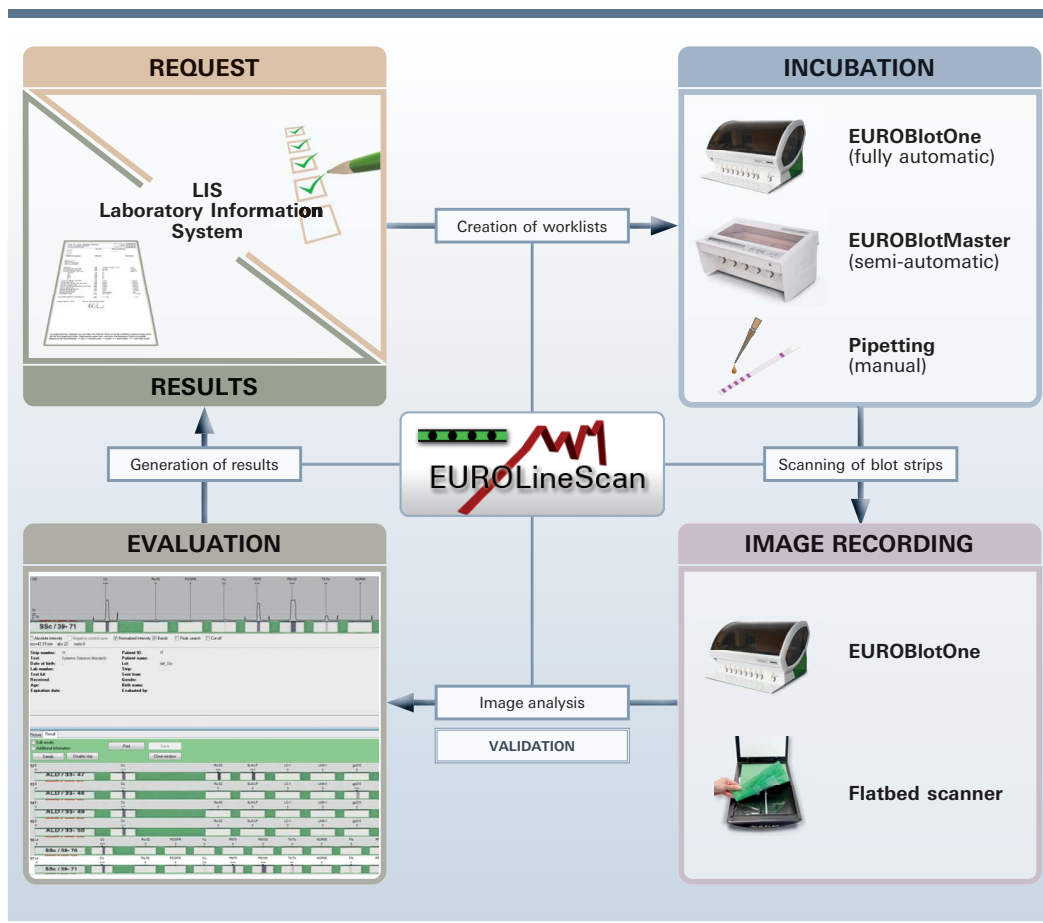

- Standardised evaluation of all EUROIMMUN blot products
- Complete results available within few minutes after incubation and digital archiving of results
- Bidirectional connection to LIS or EUROLabOffice 4.0■

Flexible and scalable automation solutions

Алматы (7273)495-231
Ангарск (3955)60-70-56
Архангельск (8182)63-90-72
Астрахань (8512)99-46-04
Барнаул (3852)73-04-60
Белгород (4722)40-23-64
Благовещенск (4162)22-76-07
Брянск (4832)59-03-52
Владивосток (423)249-28-31
Владикавказ (8672)28-90-48
Владимир (4922)49-43-18
Волгоград (844)278-03-48
Вологда (8172)26-41-59
Воронеж (473)204-51-73
Екатеринбург (343)384-55-89

Россия +7(495)268-04-70

Иваново (4932)77-34-06
Ижевск (3412)26-03-58
Иркутск (395)279-98-46
Казань (843)206-01-48
Калининград (4012)72-03-81
Калуга (4842)92-23-67
Кемерово (3842)65-04-62
Киров (8332)68-02-04
Коломна (4966)23-41-49
Кострома (4942)77-07-48
Краснодар (861)203-40-90
Красноярск (391)204-63-61
Курск (4712)77-13-04
Курган (3522)50-90-47
Липецк (4742)52-20-81

Казахстан +7(7172)727-132

Магнитогорск (3519)55-03-13
Москва (495)268-04-70
Мурманск (8152)59-64-93
Набережные Челны (8552)20-53-41
Нижний Новгород (831)429-08-12
Новокузнецк (3843)20-46-81
Ноябрьск (3496)41-32-12
Новосибирск (383)227-86-73
Омск (3812)21-46-40
Орел (4862)44-53-42
Оренбург (3532)37-68-04
Пенза (8412)22-31-16
Петрозаводск (8142)55-98-37
Псков (8112)59-10-37
Пермь (342)205-81-47

Киргизия +996(312)96-26-47

Ростов-на-Дону (863)308-18-15
Рязань (4912)46-61-64
Самара (846)206-03-16
Саранск (8342)22-96-24
Санкт-Петербург (812)309-46-40
Саратов (845)249-38-78
Севастополь (8692)22-31-93
Симферополь (3652)67-13-56
Смоленск (4812)29-41-54
Сочи (862)225-72-31
Ставрополь (8652)20-65-13
Сургут (3462)77-98-35
Сыктывкар (8212)25-95-17
Тамбов (4752)50-40-97
Тверь (4822)63-31-35

Тольятти (8482)63-91-07
Томск (3822)96-41-53
Тула (4872)33-79-87
Тюмень (3452)66-21-18
Ульяновск (8422)24-23-59
Улан-Удэ (3012)59-97-51
Уфа (347)229-48-12
Хабаровск (4212)92-98-04
Чебоксары (8352)28-53-07
Челябинск (351)202-03-61
Череповец (8202)49-02-64
Чита (3022)38-34-83
Якутск (4112)23-90-97
Ярославль (4852)69-52-93



EUROLineScan

The **EUROLineScan** system is designed for automated evaluation of incubated membrane-based test systems and administration and detailed documentation of the obtained results. The software automatically records and identifies the bands and measures their intensity. The automated evaluation of the test strips can be monitored and, if necessary, edited manually. Any changes are automatically documented. Eventually the results are archived electronically together with the image data, so that it is no longer necessary to store the potentially infectious strips. An individual result sheet can be created for each patient.


EUROLineScan does also cover the bidirectional communication with **EUROLabOffice 4.0** (laboratory management software) or a laboratory information systems (LIS) for the import of worklists and export of results.

The compact fully automatic instrument **EUROBlotOne** is designed for standardised processing of EUROIMMUN immunoblot strips (EUROLINE, EUROLINE-WB, Westernblot) – from sample recognition to the final analysis result. The samples are pipetted by the instrument and all incubation and wash steps are performed fully automatically. In the last step, the image data recorded by the integrated camera are automatically transferred to the **EUROLineScan** software.

As an alternative to **EUROBlotOne**, the immunoblot strips can also be incubated manually or using the **EUROBlotMaster** and recorded using a flatbed scanner. In this case, the automated evaluation is then also carried out by **EUROLineScan**.

EUROIMMUN is happy to assist you in the implementation of the EUROLineScan program in an existing laboratory environment.

Evaluation of incubated blot strips with EUROLineScan

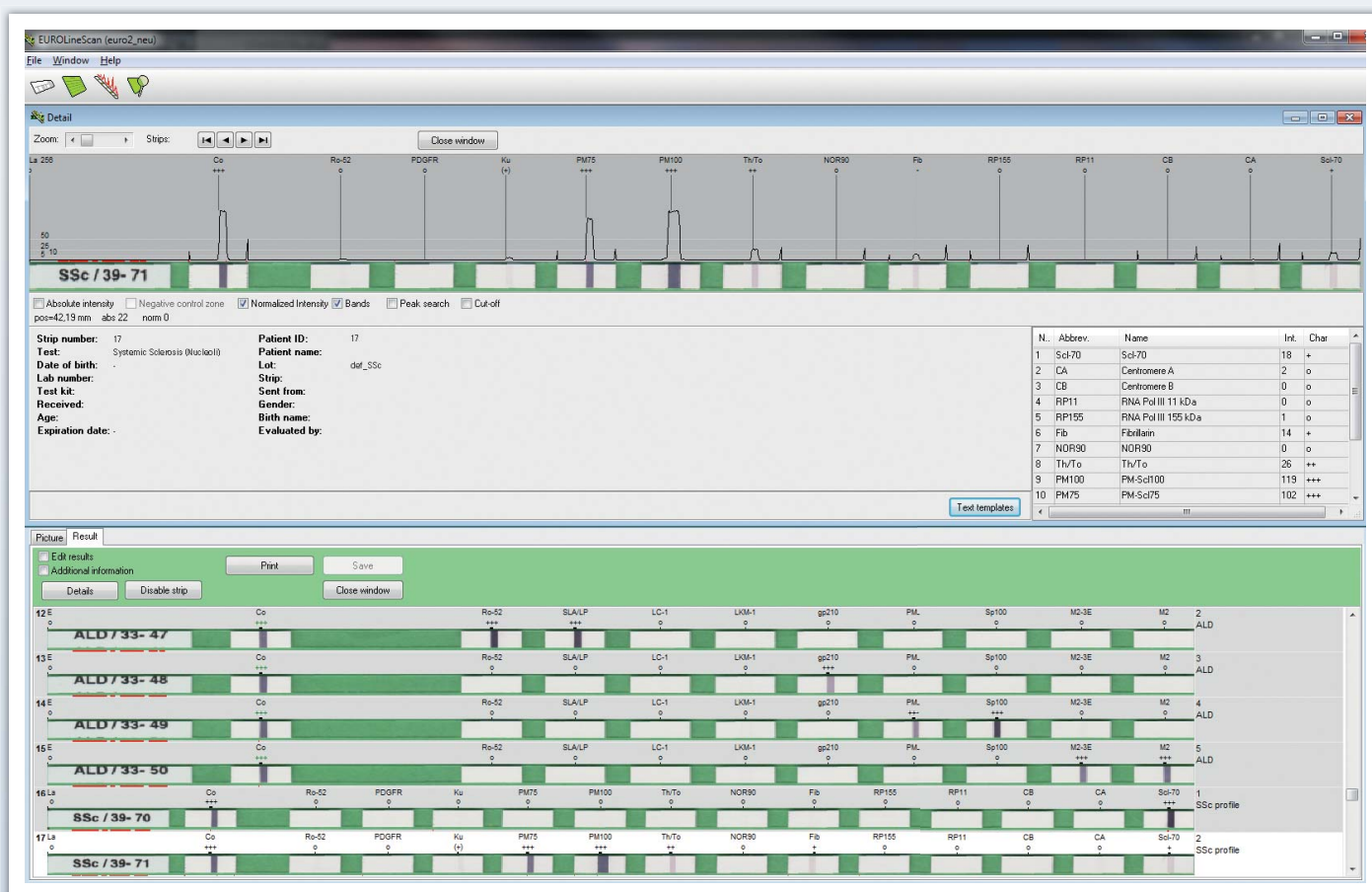


System requirements:



Computer: Pentium from 350 MHz or compatible with Windows XP, Windows Vista, Windows 7, Windows 8 or Windows 10, 512 MB RAM, approx. 1 GB free hard disc space (for the sole installation 120MB, the remaining space for graphic and patient data). **Graphic and screen resolution** at least 1024x768. **Flatbed scanner:** Canon LiDE Serie, at least 300dpi; recommended: 600dpi. **Camera system:** for direct recording of the membrane strips in the incubation tray; only available from EUROIMMUN. **Printer:** standard commercial printer is sufficient (for findings and protocols). **Barcode scanner:** Jarltech 2009 for Westernblot templates, and Honeywell Xenon 1900 or Symbol DS6707 for EUROASSAY slides.



EUROLineScan results printout (example)

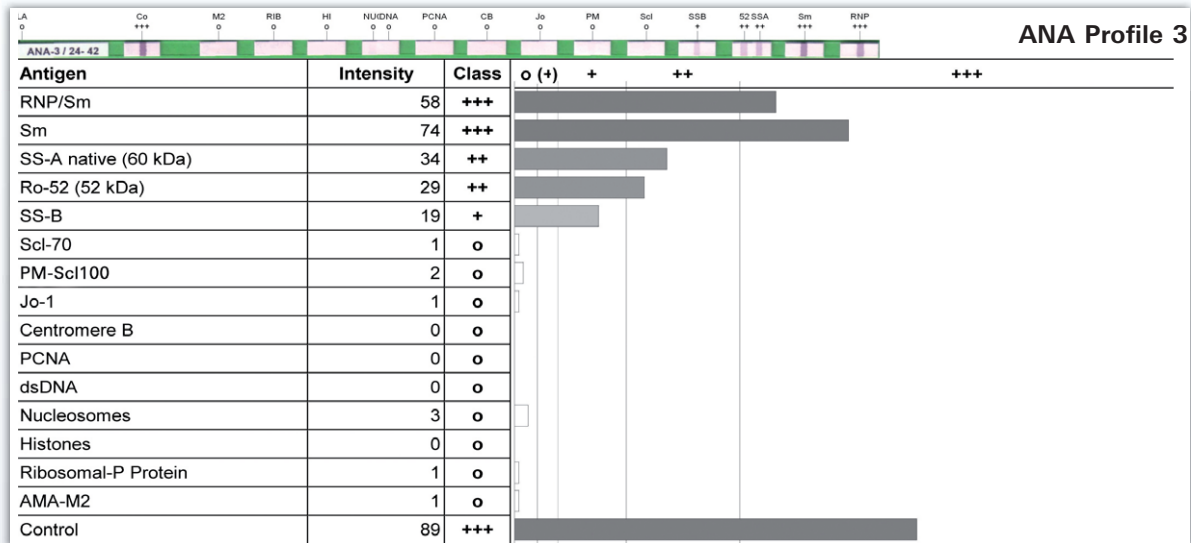

Autoimmune diagnostics

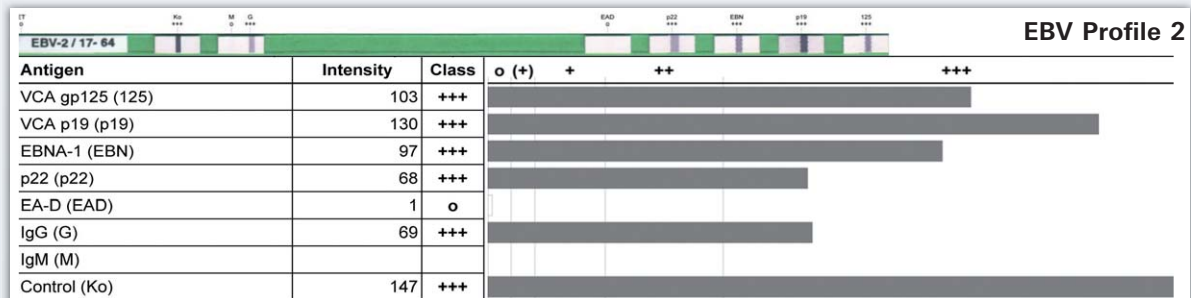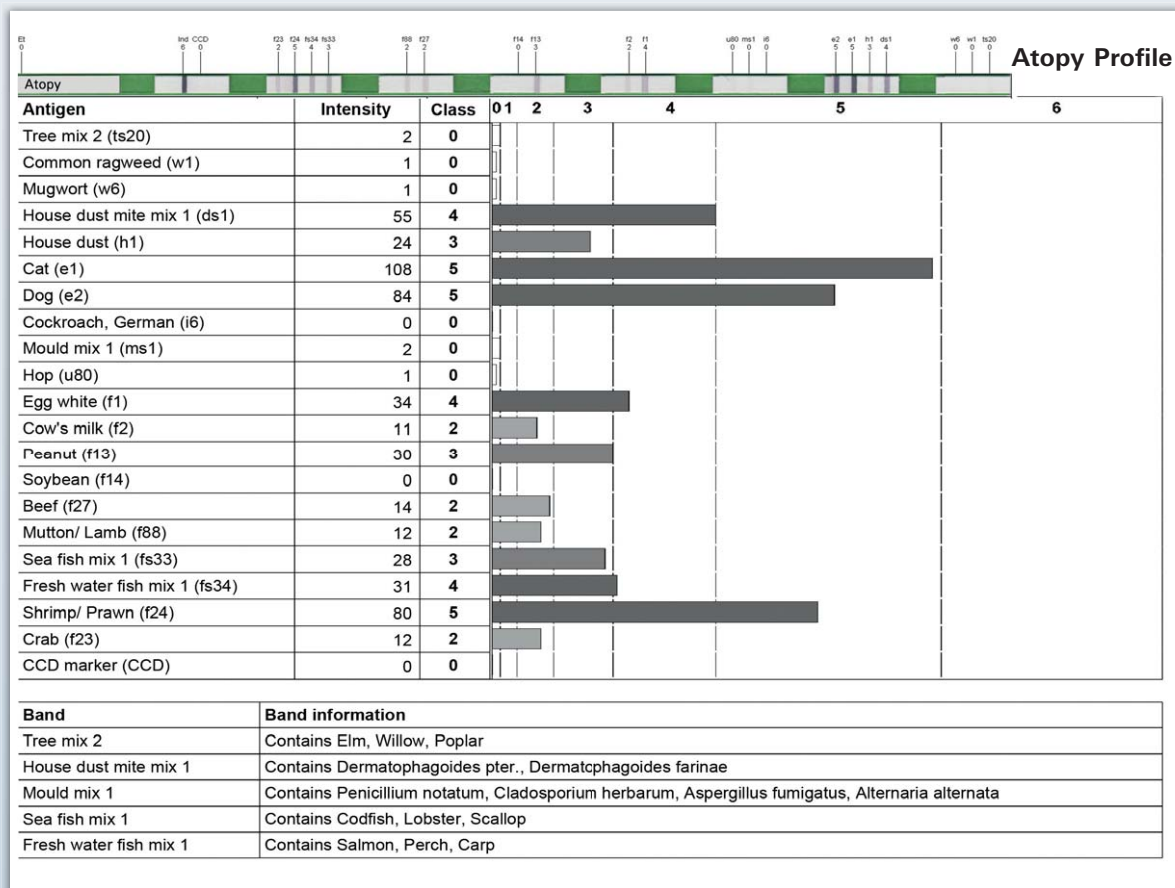Infectious
serology

Allergy diagnostics



	Product	Test	Order no.			
Autoantibody diagnostics	ANA Profi les (Mi-2α, Mi-2β, Ku, nRNP / Sm, Sm, SS-A, Ro-52, SS-B, Scl-70, PM-Scl100, PM-Scl75, Jo-1, CENP A, CENP B, PCNA, RP11, RP155, gp210, Sp100, PML, dsDNA, nucleosomes, histones, rib. P proteins, AMA-M2, RNP70, RNPA, RNPC, DFS70, Ha, Ks, Zo)	EL	DL 1590-X G	X	X	X
	Myositis Profi les (Mi-2α, Mi-2β, TIF1γ, MDA5, NXP2, SAE1, Ku, PM-Scl100, PM-Scl75, Jo-1, SRP, PL-7, PL-12, EJ, OJ, Ro-52, cN-1A, HMGCR, Ha, Ks, Zo)	EL	DL 1530-X G	X	X	X
	Systemic Sclerosis (Nucleoli) Profi les (Scl-70, CENP A, CENP B, RP11, RP155, fi brillarin, NOR-90, NVL, Th / To, PM-Scl100, PM-Scl75, Ku, PDGFR, Ro-52)	EL	DL 1532-X G	X	X	X
	Myeloperoxidase (MPO), proteinase 3 (PR3), glomerular basement membrane (GBM)	EL	DL 1200-X G	X	X	X
	Liver Profi les (AMA-M2, M2-3E (BPO), Sp100, PML, gp210, LKM-1, LC-1, SLA / LP, SS-A, Ro-52, Scl-70, CENP A, CENP B, PGDH)	EL	DL 1300-X G	X	X	X
	Gastrointestinal profi les IgA (endomysium, gliadin-analogue fusion peptide (GAF-3X), mannan (ASCA))	EL	DL 1360 A	X	X	X
	Gastrointestinal profi les IgG (endomysium, GAF-3X, PCA, intrinsic factor, ASCA)	EL	DL 1360 G	X	X	X
	AMA Profi le IgGM (AMA-M2, M2-3E, M4, M9)	EL	DL 1620-1 O	X	X	X
	Ganglioside Profi les (GM1, GM2, GM3, GD1a, GD1b, GT1b, GQ1b)	EL	DL 1130-X G, M	X ⁴	X	X
	Neuronal Antigens Profi les (amphiphysin, CV2, PNMA2 (Ma2 / Ta), Ri, Yo, Hu, recoverin, SOX1, titin, Zic4, GAD65, Tr (DNER))	EL	DL 1111-X G	X	X	X
Infection diagnostics	Coeliac Disease Profi le	EL	DL 1910 A, G	X	X	X
	Bordetella pertussis	EL	DN 2050 A, G	X	X	X
	Borrelia-RN-AT (suitable for serum and CSF)	EL	DN 2131 G, M	X	X	X
	Borrelia-RN-AT-adv (suitable for serum and CSF)	EL	DN 2131-2 M	X	X	X
	Borrelia afzelii	WB	DY 2131 G, M	X	X	X
	Borrelia afzelii plus recombinant VlsE	EL-WB	DY 2131-1 G, M	X	X	X
	Borrelia burgdorferi	WB	DY 2132 G, M	X	X	X
	Borrelia garinii	WB	DY 2134 G, M	X	X	X
	Chlamydia, human pathogen (HP)	EL-WB	DY 2190-1 A, G	X	X	X
	Cytomegaly virus	EL	DN 2570 G, M	X	X	X
	Echinococcus	EL-WB	DY 2321-1 G	X	X	X
	EBV Profi le 2 (VCA gp125, VCA p19, EBNA-1, p22, EA-D)	EL	DN 2790-2 G, M	X	X	X
	Hantavirus Profi le 1	EL	DN 278h-1 G, M	X	X	X
	Hepatitis E virus (HEV)	EL	DN 2525 A, G, M	X	X	X
	Helicobacter pylori	EL-WB	DY 2080-1 A, G	X	X	X
	Herpes simplex virus 1 (HSV-1) plus HSV-2 type-specific gG2	EL-WB	DY 2531-1 G, M	X	X	X
	Parvovirus B19	EL	DN 2580 G, M	X ⁴	X	X
	Rubella virus	WB	DY 2590 G	X	X	X
	SARS-CoV-2	EL	DN 2606-1 G	X	X	X
	TO.R.C.H. Profi le, T.O.R.C.H. 10	EL	DN 2410-4 G, M	X	X	X
	Toxocara	WB	DY 2311 G	X	X	X
	Treponema pallidum	EL	DN 2111 G, M	X	X	X
	Treponema pallidum	WB	DY 2111 G, M	X	X	X
	Treponema pallidum plus cardiolipin	EL-WB	DY 2111-1 G, M	X	X	X
	Treponema pallidum	EL	DN 2173 A, G	X	X	X
Allergy diagnostics	Yersinia enterocolitica	EL	DN 2173 A, G	X	X	X
	Yersinia enterocolitica	WB	DY 2173 A, G	X	X	X
	Veterinary diagnostics: Borrelia, dog and horse	EL	DN 2136 GC, MC, GE	X	X	X
	Veterinary diagnostics: Tick-Borne Profi le, dog and horse	EL	DN 2136-11 GC, GE	X	X	X
	Inhalation ²	EL	DP 31XX E	X	X	—
	Foods ²	EL	DP 34XX E	X	X	—
	Atopy, pediatrics ²	EL	DP 37XX E	X	X	—
	Insect venoms	EL	DP 3720 E	X	X	—
	DPA-Dx ³ Pollen ²	EL	DP 321X-1 E	X	X	—
	DPA-Dx Pediatrics	EL	DP 3812-X E	X	X	—
	DPA-Dx Milk 1	EL	DP 3510-1 E	X	X	—
	DPA-Dx Peanut 1	EL	DP 3511-1 E	X	X	—
	DPA-Dx Insect venoms ²	EL	DP 385X-X E	X	X	—

EUROLINE (EL), EUROLINE WB (EL-WB), Westernblot (WB), EUROASSAY (EA)



= using = using fl atbed scanner
EUROBlotOne



= visual

¹ under development

²including region-specific profi les³
defi ned partial allergen diagnostics⁴

no digitalization

Алматы (7273)495-231
Ангарск (3955)60-70-56
Архангельск (8182)63-90-72
Астрахань (8512)99-46-04
Барнаул (3852)73-04-60
Белгород (4722)40-23-64
Благовещенск (4162)22-76-07
Брянск (4832)59-03-52
Владивосток (423)249-28-31
Владикавказ (8672)28-90-48
Владимир (4922)49-43-18
Волгоград (844)278-03-48
Вологда (8172)26-41-59
Воронеж (473)204-51-73
Екатеринбург (343)384-55-89
Россия +7(495)268-04-70

Иваново (4932)77-34-06
Ижевск (3412)26-03-58
Иркутск (395)279-98-46
Казань (843)206-01-48
Калининград (4012)72-03-81
Калуга (4842)92-23-67
Кемерово (3842)65-04-62
Киров (8332)68-02-04
Коломна (4966)23-41-49
Кострома (4942)77-07-48
Краснодар (861)203-40-90
Красноярск (391)204-63-61
Курск (4712)77-13-04
Курган (3522)50-90-47
Липецк (4742)52-20-81
Казахстан +7(7172)727-132

Магнитогорск (3519)55-03-13
Москва (495)268-04-70
Мурманск (8152)59-64-93
Набережные Челны (8552)20-53-41
Нижний Новгород (831)429-08-12
Новокузнецк (3843)20-46-81
Ноябрьск (3496)41-32-12
Новосибирск (383)227-86-73
Омск (3812)21-46-40
Орел (4862)44-53-42
Оренбург (3532)37-68-04
Пенза (8412)22-31-16
Петрозаводск (8142)55-98-37
Псков (8112)59-10-37
Пермь (342)205-81-47
Киргизия +996(312)96-26-47

Ростов-на-Дону (863)308-18-15
Рязань (4912)46-61-64
Самара (846)206-03-16
Саранск (8342)22-96-24
Санкт-Петербург (812)309-46-40
Саратов (845)249-38-78
Севастополь (8692)22-31-93
Симферополь (3652)67-13-56
Смоленск (4812)29-41-54
Сочи (862)225-72-31
Ставрополь (8652)20-65-13
Сургут (3462)77-98-35
Сыктывкар (8212)25-95-17
Тамбов (4752)50-40-97
Тверь (4822)63-31-35

Тольятти (8482)63-91-07
Томск (3822)98-41-53
Тула (4872)33-79-87
Тюмень (3452)66-21-18
Ульяновск (8422)24-23-59
Улан-Удэ (3012)59-97-51
Уфа (347)229-48-12
Хабаровск (4212)92-98-04
Чебоксары (8352)28-53-07
Челябинск (351)202-03-61
Череповец (8202)49-02-64
Чита (3022)38-34-83
Якутск (4112)23-90-97
Ярославль (4852)69-52-93

eoа@nt-rt.ru || <https://euroimmun.nt-rt.ru/>